

## SAFETY FEATURES

The FabRoc® heating system has been designed for safe and reliable use. It has been tested to comply with European safety regulations and the power system has been CE marked. Safety devices are included to ensure the unit is rendered safe, even under fault conditions.

DO NOT replace or interfere with the electrical components in your rug or the power pack and charger unit.

## WASHING INSTRUCTIONS

1. Please make sure the power packs are disconnected from ExoGlo rug before washing.
2. To wash: hose-down or hand wash with soapflakes only.
3. Drip dry.
4. Do not dry clean.

## WARRANTY

FabRoc® has a lifetime guarantee – so in the unlikely event your ExoGlo rug does not heat at any time in the future just contact us and we will repair or replace it (proof of purchase is required for warranty).

*Note: the warranty does not cover intentional alterations, accidental damage or improper cleaning.*

## TROUBLESHOOTING

### What happens if my ExoGlo heated equine rug doesn't get warm?

Firstly, check that you have the rug connected to a power source.

Ensure the portable power packs are charged. This can be done by firstly plugging the power packs into the associated charger. If the red light on the charger comes on and stays on, the power pack needs to be charged. If the power pack is charged, check that the rug plug is correctly attached to the powerpack.


Should you have any questions about your heated products, please contact:

Customer Services Dept  
EXO2  
Caledonian House  
Phoenix Crescent  
Strathclyde Business Park  
Bellshill ML4 3NJ  
Tel: +44 (0)1698 746347  
Email: sales@exo2.co.uk



The Heat Inside®

www.exo2.co.uk


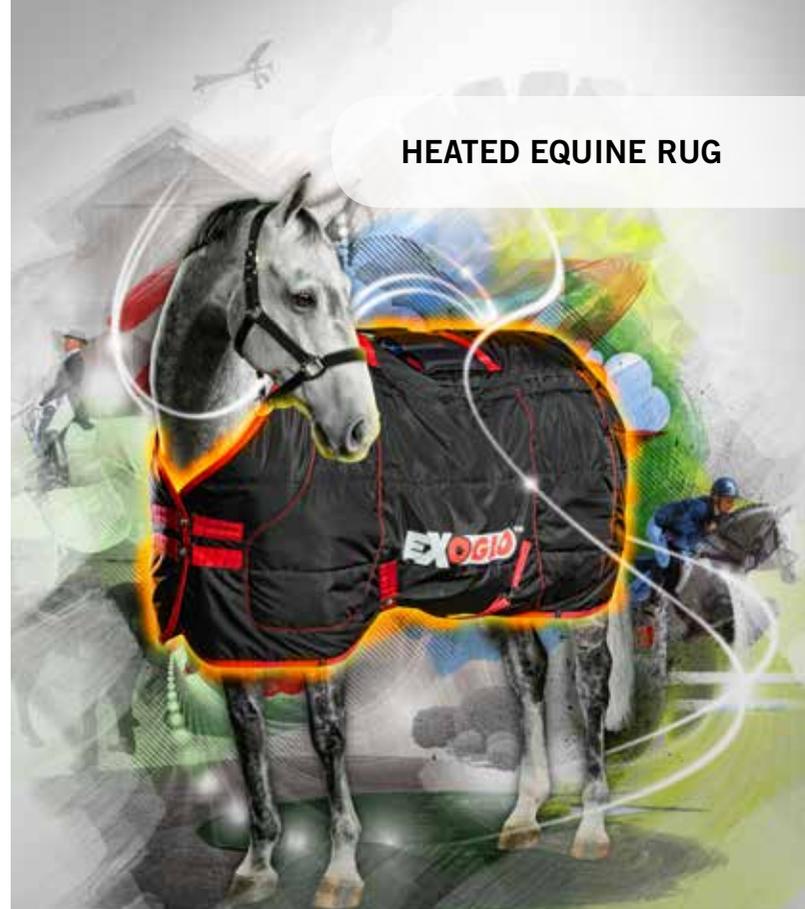


HEATED EQUINE RUG

**EXOGLO™**


## The ExoGlo Heated Equine Rug has:

- **4 heating panels – one on each quad.**
- **A total FabRoc® heated area of 4860cm sq.**
- **FabRoc® set to a maximum temperature of 55°C.**
- **No hotspots - FabRoc® exhibits a positive temperature coefficient guaranteeing uniform heat across FabRoc® panels.**
- **FabRoc® FIR technology - heats muscles to a depth of 4cm not just the surface of the skin.**

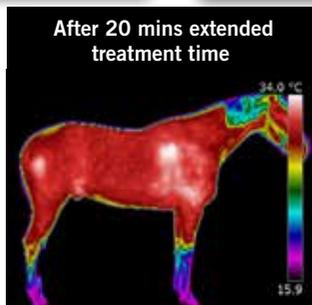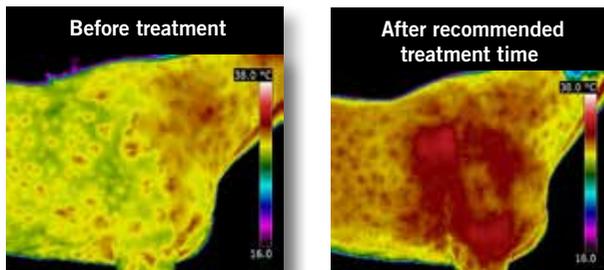

## Directions for use:

1. Place the ExoGlo rug on the horse.
2. Fasten up front straps and cross over belly straps.
3. Place the four power packs in the power pack pouch.
4. The power pack pouch has velcro on the underside. Position the power pack pouch on the velcro on the top of the rug. The power pack pouch should sit in a similar position to a saddle.
5. Attach the power packs to the ExoGlo rug with the wires positioned in front of the power pack pouch. The colours and numbers on the ExoGlo rug and the front colour panel of the power pack correspond one to each quad of the rug. The remote controls also have a coloured number on the reverse.
6. You can either use the remote controls or the buttons on the power packs to turn on the power packs and to regulate the temperature.
7. The power packs have three temperature settings.
  - 1 = low heat (approx. 18 degrees)
  - 2 = medium heat (approx. 36 degrees)
  - 3 = high heat (approx. 55 degrees).The temperature setting will show on the power pack display with a "." Next to the number e.g. "1." "2." "3."
8. The display on the battery will automatically show between 1 and 9 once the temperature level has been chosen. This number represents the capacity level available 1 = very low capacity, 9 = full capacity.
9. To turn the power packs off you can either use the red button on the remote or the off button on the power pack.

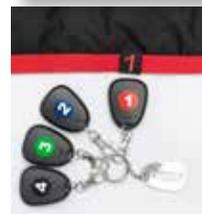

10. Each power pack heats one quad. This gives the flexibility for each of the four quads to be heated independently. All quads can have the temperature raised or lowered independently of the others by using the settings on the remote controls or power packs.
11. Quads can be heated for a maximum of four hours on full power and nine hours on low power using one ExoGlo power pack.
12. Recommended treatment time pre exercise or massage treatment is 20 minutes on full power (setting 3).
13. For increased flexibility when travelling, at a show or for treatment of a specific injury you can heat one quad for longer periods by connecting the wire connector for that quad (use colour number code to identify each quad) to a different power pack.

## TO CHARGE THE POWER PACKS

1. Disconnect the power pack from the ExoGlo rug and remove from the power pack pouch.
2. Connect charger unit to power pack. The light on the front of the charger will be green when the charger is ready to charge power pack. Once a depleted power pack is connect to the charger the light will turn red while the charger is checking on the level of battery power. The red light stays on continuously while the power pack is being charged. The red light turns to green when fully charged.
3. The charger supplied has been chosen specifically for the power pack supplied. No other charger must be used.

**EXOGLO™**